

## Taulukko 3.2.Ristireaktiot kauppanimen mukaan

| Drugs of Abuse Assays Cross-Reactivity Manual by Trade Name |   |   |
|---|---|---|
| COMPOUND (Trade Name)                                       | COMPOUND (Generic Name)                           | RESULTS   |
| Acephen   | Acetaminophen (See also Paracetamol)              | Non-reactive  |
| Aceta   | Acetaminophen (See also Paracetamol)              | Non-reactive  |
| Actinac   | Hydrocortisone                                    | Non-reactive  |
| Adalat  | Nifedipine  | Non-reactive  |
| Adam (slang)  | Methylenedioxyamphetamine (MDMA)                  | Positive for Methamphetamine (MET) and Ecstasy (MDMA) |
| Adderall  | Dexamphetamine Sulphate                           | Positive for Amphetamine (AMP)                        |
| Adderall XR   | Dexamphetamine Sulphate                           | Positive for Amphetamine (AMP)                        |
| AeroBec   | Beclometasone                                     | Non-reactive  |
| AeroBec Forte   | Beclometasone                                     | Non-reactive  |
| Aerocrom  | Salbutamol  | Non-reactive  |
| Aerolin   | Salbutamol  | Non-reactive  |
| Afrin   | Oxymetazoline                                     | Non-reactive  |
| Afrinol   | Pseudoephedrine                                   | Non-reactive  |
| Airomir   | Salbutamol  | Non-reactive  |
| Aleve   | Naproxen  | Non-reactive  |
| Alisone   | Aluminum Hydroxide                                | Non-reactive  |
| Allerfen  | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Alphaderm   | Hydrocortisone                                    | Non-reactive  |
| Alphosyl-HC   | Hydrocortisone                                    | Non-reactive  |
| Alu-Cap   | Aluminum Hydroxide                                | Non-reactive  |
| Amoram  | Amoxicillin or Amoxicillin                        | Non-reactive  |
| Amoxil  | Amoxicillin or Amoxicillin                        | Non-reactive  |
| Amplan  | Pericyazine                                       | Non-reactive  |
| Amytal  | Amobarbital                                       | Positive for Barbiturates (BAR)                       |
| Anabact   | Metronidazole                                     | Non-reactive  |
| Anadin  | Acetylsalicylic acid                              | Non-reactive  |
| Anasin  | Acetylsalicylic acid                              | Non-reactive  |
| Andropatch  | Testosterone                                      | Non-reactive  |
| Anergan   | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Angettes  | Aspirin   | Non-reactive  |
| Angitak   | Isosorbide Dinitrate                              | Non-reactive  |
| Anhydrol Forte  | Aluminum Chloride Hexahydrate                     | Non-reactive  |
| Antaxone  | Naltrexone  | Non-reactive  |
| Antinaus  | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Anugestic-HC  | Hydrocortisone                                    | Non-reactive  |
| Anusol-HC   | Hydrocortisone                                    | Non-reactive  |
| Anxitol   | Medazepam   | Positive for Benzodiazepines (BZD)                    |
| Aolept  | Pericyazine                                       | Non-reactive  |
| Apacet  | Acetaminophen (See also Paracetamol)              | Non-reactive  |
| Apamin  | Pericyazine                                       | Non-reactive  |
| Aprinox   | Bendrofluazide                                    | Non-reactive  |
| Aprobit   | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Arpimycin   | Erythromycin                                      | Non-reactive  |
| ASA   | Aspirin   | Non-reactive  |
| Asasantin   | Aspirin   | Non-reactive  |
| Asilone   | Dimethicone                                       | Non-reactive  |
| Asmabec   | Beclometasone                                     | Non-reactive  |
| Asmasal   | Salbutamol  | Non-reactive  |
| Aspirin   | Acetylsalicylic acid                              | Non-reactive  |
| Astramorph  | Morphine  | Positive for Opiates (MOR, OPI)                       |
| Atarax  | Hydroxyzine Hydrochloride                         | Non-reactive  |
| Ativan  | Lorazepam   | Positive for Benzodiazepines (BZD)                    |
| Augmentin   | Amoxicillin or Amoxicillin                        | Non-reactive  |
| Augmentin   | Co-Amoxiclav                                      | Non-reactive  |
| Augmentin Duo   | Co-Amoxiclav                                      | Non-reactive  |
| Avomine   | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Bactrigras  | Chlorhexidine Gluconate                           | Non-reactive  |
| Bayer Aspirin   | Aspirin   | Non-reactive  |
| Baymethzine   | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)          |
| Beclazone   | Beclometasone                                     | Non-reactive  |
| Becloforte  | Beclometasone                                     | Non-reactive  |
| Becodisks   | Beclometasone                                     | Non-reactive  |
| Beconase  | Beclometasone                                     | Non-reactive  |
| Becotide  | Beclometasone                                     | Non-reactive  |
| Benadryl  | Diphenhydramine                                   | Non-reactive  |
| Benzymycin  | Erythromycin                                      | Non-reactive  |
| Beta-adlat  | Atenolol  | Non-reactive  |
| Beta-Progane  | Propranolol                                       | Non-reactive  |
| BiNovum   | Norethindrone                                     | Non-reactive  |
| Bradosol  | Benzalkonium                                      | Non-reactive  |
| Brevinor  | Norethindrone                                     | Non-reactive  |
| Briantum  | Delorazepam                                       | Positive for Benzodiazepines (BZD)                    |
| Britoflex   | Lofexidine  | Non-reactive  |
| Brufen  | Ibuprofen   | Non-reactive  |
| Bufferin  | Acetylsalicylic acid                              | Non-reactive  |
| Busocopan   | Hyoscine butylbromide                             | Non-reactive  |
| Butisol   | Butabarbital                                      | Positive for Barbiturates (BAR)                       |
| Calmurid-HC   | Hydrocortisone                                    | Non-reactive  |
| Canestan ear/skin/vaginitis/HC                              | Canestan  | Non-reactive  |
| Canesten-HC   | Hydrocortisone                                    | Non-reactive  |
| Caprin  | Acetylsalicylic acid                              | Non-reactive  |
| Caprin  | Aspirin   | Non-reactive  |
| Carace  | Lisinopril HCl                                    | Non-reactive  |
| Cedocard  | Isosorbide Dinitrate                              | Non-reactive  |
| Centrax   | Prazepam  | Positive for Benzodiazepines (BZD)                    |
| Ceporex   | Cephalexin  | Non-reactive  |
| Chlorohex   | Chlorhexidine Gluconate                           | Non-reactive  |
| Cidomycin   | Gentamicin Sulphate                               | Non-reactive  |
| Cipramil  | Citalopram  | Non-reactive  |
| Claritin  | Loratadine  | Non-reactive  |
| Climagest   | Norethindrone                                     | Non-reactive  |
| Climesse  | Norethindrone                                     | Non-reactive  |
| Clonopin  | Clonazepam  | Positive for Benzodiazepines (BZD)                    |
| Co-codamol  | Paracetamol (Acetaminophen) /Codeine Preparations | Positive for Opiates (MOR, OPI)                       |
| Codafen   | Ibuprofen   | Non-reactive  |
| Codafen   | Paracetamol (Acetaminophen) /Codeine Preparations | Positive for Opiates (MOR, OPI)                       |
| Codafen Continus  | Codeine Phosphate                                 | Positive for Opiates (OP/MOR)                         |
| Codeine Linctus   | Codeine Phosphate                                 | Positive for Opiates (OP/MOR)                         |
| Co-dydramol   | Paracetamol (Acetaminophen) /Codeine Preparations | Positive for Opiates (MOR, OPI)                       |
| Co-fluampicil   | Flucloxacillin                                    | Non-reactive  |

| COMPOUND (Trade Name)  | COMPOUND (Generic Name)              | RESULTS   |
|------------------------|--------------------------------------|---|
| Co-garetic             | Bendrofluazide                       | Non-reactive  |
| Colioam                | Hydrocortisone                       | Non-reactive  |
| Combicillin            | Penicillin                           | Non-reactive  |
| Combivent              | Salbutamol                           | Non-reactive  |
| Condrotec              | Naproxen                             | Non-reactive  |
| Co-proximal            | Dextropropoxyphene                   | Positive for Propoxyphene (PPX)                       |
| Cordsodyl              | Chlorhexidine Gluconate              | Non-reactive  |
| Corgard                | Nadolol                              | Non-reactive  |
| Corgaretic             | Nadolol                              | Non-reactive  |
| Corgartic              | Nadolol                              | Non-reactive  |
| Co-tenidone            | Atenolol                             | Non-reactive  |
| Coumadin               | Warfarin Sodium                      | Non-reactive  |
| Crystapen              | Benzylpenicillin                     | Non-reactive  |
| CX Powder              | Chlorhexidine Gluconate              | Non-reactive  |
| Cyclimorph             | Morphine                             | Positive for Opiates (MOR, OPI)                       |
| Daktacort              | Hydrocortisone                       | Non-reactive  |
| Dapacen                | Acetaminophen (See also Paracetamol) | Non-reactive  |
| Darvocet               | Dextropropoxyphene                   | Positive for Propoxyphene (PPX)                       |
| Darvon                 | Dextropropoxyphene                   | Positive for Propoxyphene (PPX)                       |
| Decadron               | Dexamethasone                        | Non-reactive  |
| Delta-cortef           | Prednisolone                         | Non-reactive  |
| Demerol                | Meperidine                           | Non-reactive  |
| Demetrin               | Prazepam                             | Positive for Benzodiazepines (BZD)                    |
| Depakene               | Sodium Valproate                     | Non-reactive  |
| Depakote               | Sodium Valproate                     | Non-reactive  |
| Depixol                | Flupentixol Decanoate                | Non-reactive  |
| Dermol                 | Chlorhexidine Gluconate              | Non-reactive  |
| Desoxyn                | d-Methamphetamine HCL                | Positive for Methamphetamine (MET)                    |
| Desyrel                | Trazodone                            | Non-reactive  |
| Dexa-Rhinaspray Duo    | Dexamethasone                        | Non-reactive  |
| Dexedrine              | Dexamphetamine Sulphate              | Positive for Amphetamine (AMP)                        |
| DHC Continus           | Dihydrocodeine                       | Positive for Opiates (OPI/MOR)                        |
| Diazemuls              | Diazepam                             | Positive for Benzodiazepines (BZD)                    |
| Dicloflex              | Diclofenac Sodium                    | Non-reactive  |
| Diclomax               | Diclofenac Sodium                    | Non-reactive  |
| Diffiam                | Benzydamine HCl                      | Non-reactive  |
| Dilantin               | Phenytoin                            | Non-reactive  |
| Dilaudid               | Hydromorphone                        | Positive for Opiates (OPI/MOR)                        |
| Dimetapp               | Brompheniramine                      | Non-reactive  |
| Dimotane               | Brompheniramine                      | Non-reactive  |
| Dimotapp               | Brompheniramine                      | Non-reactive  |
| Diocalm                | Koalin and Morphine Mixture          | Positive for Opiates (OPI/MOR)                        |
| Dioderm                | Hydrocortisone                       | Non-reactive  |
| Diprozin               | Promethazine                         | Non-reactive  |
| Disipil                | Orphenadrine                         | Non-reactive  |
| Disprin                | Acetylsalicylic acid                 | Non-reactive  |
| Dolophine              | Methadone Hydrochloride              | Positive for Methadone (MTD)                          |
| Donnatal               | Phenobarbital                        | Positive for Barbiturates (BAR)                       |
| Drichlor               | Aluminum Chloride Hexahydrate        | Non-reactive  |
| Duovent                | Salbutamol                           | Non-reactive  |
| Duphalec               | Lactulose                            | Non-reactive  |
| Duramorph              | Morphine                             | Positive for Opiates (MOR, OPI)                       |
| Dyspamet               | Cimetidine                           | Non-reactive  |
| E (slang)              | Methylenedioxyamphetamine (MDMA)     | Positive for Methamphetamine (MET) and Ecstasy (MDMA) |
| Econacort              | Hydrocortisone                       | Non-reactive  |
| Econopred              | Prednisolone                         | Non-reactive  |
| Ecotrin                | Acetylsalicylic acid                 | Non-reactive  |
| Ecstasy (slang)        | Methylenedioxyamphetamine (MDMA)     | Positive for Methamphetamine (MET) and Ecstasy (MDMA) |
| Efcortesel             | Hydrocortisone                       | Non-reactive  |
| Effexor                | Venlafaxine                          | Potential Positive for PCP                            |
| Effexor XL             | Venlafaxine                          | Potential Positive for PCP                            |
| Efortelan              | Hydrocortisone                       | Non-reactive  |
| Elavil                 | Amitriptyline                        | Positive for Tricyclic Antidepressants (TCA)          |
| Elleste Duet           | Norethindrone                        | Non-reactive  |
| Eltroxin               | Thyroxine Sodium                     | Non-reactive  |
| Elyzol                 | Metronidazole                        | Non-reactive  |
| Empirin                | Acetylsalicylic acid                 | Non-reactive  |
| Entersan               | Koalin and Morphine Mixture          | Positive for Opiates (OPI/MOR)                        |
| Epanutin               | Phenytoin                            | Non-reactive  |
| Epilim                 | Sodium Valproate                     | Non-reactive  |
| Epitard                | Phenytoin                            | Non-reactive  |
| Erycane                | Erythromycin                         | Non-reactive  |
| Erymax                 | Erythromycin                         | Non-reactive  |
| Erythrocin             | Erythromycin                         | Non-reactive  |
| Erythroped             | Erythromycin                         | Non-reactive  |
| Estracombi             | Norethindrone                        | Non-reactive  |
| Eurax-HC               | Hydrocortisone                       | Non-reactive  |
| Eve (slang)            | Methylenedioxyamphetamine (MDA)      | Positive for Amphetamine (AMP) and Ecstasy (MDMA)     |
| Evorel                 | Norethindrone                        | Non-reactive  |
| Excedrin               | Aspirin                              | Non-reactive  |
| Excedrin (combination) | Acetaminophen (See also Paracetamol) | Non-reactive  |
| Excedrin (combination) | Acetylsalicylic acid                 | Non-reactive  |
| Fancorsil              | Dimethicone                          | Non-reactive  |
| Fenbid                 | Ibuprofen                            | Non-reactive  |
| Fenergan               | Promethazine                         | Positive for Tricyclic Antidepressants (TCA)          |
| Feverall               | Acetaminophen (See also Paracetamol) | Non-reactive  |
| Fillair                | Beclometasone                        | Non-reactive  |
| Fioricet               | Butalbital                           | Positive for Barbiturates (BAR)                       |
| Fiorinal               | Butalbital                           | Positive for Barbiturates (BAR)                       |
| Flagyl                 | Metronidazole                        | Non-reactive  |
| Flexin continus        | Indomethacin                         | Non-reactive  |
| Floxapen               | Flucloxacillin                       | Non-reactive  |
| Fluanxol               | Flupentixol Decanoate                | Non-reactive  |
| Fortagesic             | Paracetamol (Acetaminophen)          | Non-reactive  |
| Frisium                | Clobazam                             | Positive for Benzodiazepines (BZD)                    |
| Fucibet                | Fusidic Acid (Sodium Fusidate)       | Non-reactive  |
| Fucidin                | Fusidic Acid (Sodium Fusidate)       | Non-reactive  |
| Fucithalamic           | Fusidic Acid (Sodium Fusidate)       | Non-reactive  |
| Galcodine              | Codeine Phosphate                    | Positive for Opiates (OPI/MOR)                        |
| Galenphol              | Pholcodine                           | Positive for Opiates (MOR, OPI)                       |
| Galpseud               | Chlorpheniramine Maleate             | Non-reactive  |
| Gamanil                | Lofepamine                           | Positive for Tricyclic Antidepressants (TCA)          |

| COMPOUND (Trade Name) | COMPOUND (Generic Name)                           | RESULTS   |
|-----------------------|---|---|
| Gastrocote            | Aluminum Hydroxide                                | Non-reactive                                      |
| Gaviscon              | Magnesium Alginate                                | Non-reactive                                      |
| Gaviscon              | Mannitol  | Non-reactive                                      |
| Genticin              | Gentamicin Sulphate                               | Non-reactive                                      |
| Gentisone             | Gentamicin Sulphate                               | Non-reactive                                      |
| Gregoderm             | Hydrocortisone                                    | Non-reactive                                      |
| Halcion               | Triazolam   | Positive for Benzodiazepines (BZD)                |
| Haymine               | Chlorpheniramine Maleate                          | Non-reactive                                      |
| Heliclear             | Amoxicillin or Amoxycillin                        | Non-reactive                                      |
| Heliclear             | Lansoprazole                                      | Non-reactive                                      |
| Hibicet               | Chlorhexidine Gluconate                           | Non-reactive                                      |
| Hibiscrub             | Chlorhexidine Gluconate                           | Non-reactive                                      |
| Hibisol               | Chlorhexidine Gluconate                           | Non-reactive                                      |
| Hibitane              | Chlorhexidine Gluconate                           | Non-reactive                                      |
| Hydrocortistab        | Hydrocortisone                                    | Non-reactive                                      |
| Hydrocortone          | Hydrocortisone                                    | Non-reactive                                      |
| Hydrostat             | Hydromorphone                                     | Positive for Opiates (OPI/MOR)                    |
| Ibugel                | Ibuprofen   | Non-reactive                                      |
| Ibuspray              | Ibuprofen   | Non-reactive                                      |
| Ilosone               | Erythromycin                                      | Non-reactive                                      |
| Imodium               | Loperamide HCl                                    | Non-reactive                                      |
| Inderal               | Propranolol                                       | Non-reactive                                      |
| Inderetic             | Propranolol                                       | Non-reactive                                      |
| Inderex               | Bendrofluazide                                    | Non-reactive                                      |
| Inderex               | Propranolol                                       | Non-reactive                                      |
| Indertic              | Bendrofluazide                                    | Non-reactive                                      |
| Indocid               | Indomethacin                                      | Non-reactive                                      |
| Indomod               | Indomethacin                                      | Non-reactive                                      |
| Inflamase             | Prednisolone                                      | Non-reactive                                      |
| INH                   | Isoniazide  | Non-reactive                                      |
| Instillagel           | Chlorhexidine Gluconate                           | Non-reactive                                      |
| Intron A              | Interferon Alfa                                   | Non-reactive                                      |
| Ipecac                | Ammonia/Ipecacuanha                               | Non-reactive                                      |
| Iryakin               | Pericyazine                                       | Non-reactive                                      |
| Isocard               | Isosorbide Dinitrate                              | Non-reactive                                      |
| Isoket                | Isosorbide Dinitrate                              | Non-reactive                                      |
| Isordil               | Isosorbide Dinitrate                              | Non-reactive                                      |
| Isotrexin             | Erythromycin                                      | Non-reactive                                      |
| Kalten                | Atenolol  | Non-reactive                                      |
| Kapake                | Codeine Phosphate                                 | Positive for Opiates (OPI/MOR)                    |
| Kapake                | Paracetamol (Acetaminophen) /Codeine Preparations | Positive for Opiates (MOR, OPI)                   |
| Keflex                | Cephalexin  | Non-reactive                                      |
| Kliofem               | Norethindrone                                     | Non-reactive                                      |
| Kliovance             | Norethindrone                                     | Non-reactive                                      |
| Klonopin              | Clonazepam  | Positive for Benzodiazepines (BZD)                |
| Kolanticon            | Aluminum Hydroxide                                | Non-reactive                                      |
| Kolanticon            | Dicyclomine                                       | Non-reactive                                      |
| Lactobinate           | Erythromycin                                      | Non-reactive                                      |
| Lactugal              | Lactulose   | Non-reactive                                      |
| Lamical               | Lamotrigine                                       | Non-reactive                                      |
| Largactil             | Chlorpromazine                                    | Non-reactive                                      |
| Lentizol              | Amitriptyline                                     | Positive for Tricyclic Antidepressants (TCA)      |
| Lerisum               | Medazepam   | Positive for Benzodiazepines (BZD)                |
| Lexotan               | Bromazepam  | Positive for Benzodiazepines (BZD)                |
| Librium               | Chlordiazepoxide                                  | Positive for Benzodiazepines (BZD)                |
| Lipitor               | Atorvastatin                                      | Non-reactive                                      |
| Locoid                | Hydrocortisone-17-butyrate                        | Non-reactive                                      |
| Locoid C              | Hydrocortisone-17-butyrate                        | Non-reactive                                      |
| Loestrin              | Norethindrone                                     | Non-reactive                                      |
| Lomont                | Lofepamine  | Positive for Tricyclic Antidepressants (TCA)      |
| Lomotil               | Co-Phenotrope (atropine/diphenoxylate)            | Non-reactive                                      |
| Lomotil               | Diphenoxylate with Atropine                       | Non-reactive                                      |
| Loperagen             | Loperamide HCl                                    | Non-reactive                                      |
| Loratab               | Hydrocodone                                       | Positive for Opiates (OPI/MOR)                    |
| Lorcet                | Hydrocodone                                       | Positive for Opiates (OPI/MOR)                    |
| Love Drug (slang)     | Methylenedioxymphetamine (MDA)                    | Positive for Amphetamine (AMP) and Ecstasy (MDMA) |
| Luminal               | Phenobarbitone (see also Phenobarbital)           | Positive for Barbiturates (BAR)                   |
| Maalox                | Aluminum Hydroxide                                | Non-reactive                                      |
| Maalox TC             | Aluminum Hydroxide                                | Non-reactive                                      |
| Magnapen              | Flucloxacillin                                    | Non-reactive                                      |
| Magnapen              | Co-Fluampicil                                     | Non-reactive                                      |
| Marevan               | Warfarin Sodium                                   | Non-reactive                                      |
| Maxidex               | Dexamethasone                                     | Non-reactive                                      |
| Maxidrol              | Dexamethasone                                     | Non-reactive                                      |
| Medacepan             | Medazepam   | Positive for Benzodiazepines (BZD)                |
| Medinex               | Diphenhydramine                                   | Non-reactive                                      |
| Mefoxin               | Penicillin  | Non-reactive                                      |
| Mellari               | Thioridazine Hydrochloride                        | Non-reactive                                      |
| Merbentyl             | Dicyclomine                                       | Non-reactive                                      |
| Methadose             | Methadone Hydrochloride                           | Positive for Methadone (MTD)                      |
| Methamprex            | d-Methamphetamine HCL                             | Positive for Methamphetamine (MET)                |
| Methazine             | Promethazine                                      | Positive for Tricyclic Antidepressants (TCA)      |
| Methedrine            | d-Methamphetamine HCL                             | Positive for Methamphetamine (MET)                |
| Metrogel              | Metronidazole                                     | Non-reactive                                      |
| Metrolyl              | Metronidazole                                     | Non-reactive                                      |
| Metrotop              | Metronidazole                                     | Non-reactive                                      |
| Micronor              | Norethindrone                                     | Non-reactive                                      |
| Migraleve             | Codeine Phosphate                                 | Positive for Opiates (OPI/MOR)                    |
| Mildison lipocream    | Hydrocortisone                                    | Non-reactive                                      |
| Minims                | Dexamethasone                                     | Non-reactive                                      |
| Mogadon               | Nitrazepam  | Positive for Benzodiazepines (BZD)                |
| Molipaxin             | Trazodone   | Non-reactive                                      |
| Morcap                | Morphine  | Positive for Opiates (MOR, OPI)                   |
| Morphine Sulfate      | Morphine  | Positive for Opiates (MOR, OPI)                   |
| Motifene              | Diclofenac Sodium                                 | Non-reactive                                      |
| Motrin                | Ibuprofen   | Non-reactive                                      |
| MS Contin             | Morphine  | Positive for Opiates (MOR, OPI)                   |
| Mucogel               | Aluminum Hydroxide                                | Non-reactive                                      |
| Myambutol             | Ethambutol  | Non-reactive                                      |
| Nalorex               | Naltrexone  | Non-reactive                                      |
| Napratec              | Naproxen  | Non-reactive                                      |
| Naprosen              | Naproxen  | Non-reactive                                      |

| COMPOUND (Trade Name) | COMPOUND (Generic Name)                           | RESULTS  |
|-----------------------|---|--|
| Naprosyn              | Naproxen  | Non-reactive   |
| Narcan                | Naloxone  | Non-reactive   |
| Narcotine             | Noscapine   | Non-reactive   |
| Naseptin              | Chlorhexidine Gluconate                           | Non-reactive   |
| Nasobec               | Beclometasone                                     | Non-reactive   |
| Nemactil              | Pericyazine                                       | Non-reactive   |
| Nembutal              | Pentobarbital                                     | Positive for Barbiturates (BAR)  |
| Neo-cortef            | Hydrocortisone                                    | Non-reactive   |
| Neo-Naclax            | Bendrofluazide                                    | Non-reactive   |
| Neo-Naclax-K          | Bendrofluazide                                    | Non-reactive   |
| Neo-synephrine        | Oxymetazoline                                     | Non-reactive   |
| Neulactil             | Pericyazine                                       | Non-reactive   |
| Neuratop              | Metronidazole                                     | Non-reactive   |
| Nicoderm              | Nicotine  | Non-reactive   |
| Nicorette             | Nicotine  | Non-reactive   |
| Nicotinell            | Nicotine  | Non-reactive   |
| Nicotrol              | Nicotine  | Non-reactive   |
| Nifar                 | Nifedipine  | Non-reactive   |
| Nifecard              | Nifedipine  | Non-reactive   |
| Nif-ten               | Nifedipine  | Non-reactive   |
| Niquitin              | Nicotine  | Non-reactive   |
| No known trade names  | Allobarbital                                      | Positive for Barbiturates (BAR)  |
| No known trade names  | Alphenol  | Positive for Barbiturates (BAR)  |
| No known trade names  | d/l-Octopamine                                    | Non-reactive   |
| Nobritol              | Medazepam   | Positive for Benzodiazepines (BZD)   |
| Nobrium               | Medazepam   | Positive for Benzodiazepines (BZD)   |
| Noctamide             | Lormetazepam                                      | Positive for Benzodiazepines (BZD)   |
| Norflex               | Orphenadrine                                      | Non-reactive   |
| Norgesic              | Orphenadrine                                      | Non-reactive   |
| Noriday               | Norethindrone                                     | Non-reactive   |
| Norimide              | Loperamide HCl                                    | Non-reactive   |
| Norimin               | Norethindrone                                     | Non-reactive   |
| Norinyl               | Norethindrone                                     | Non-reactive   |
| Noristat              | Norethindrone                                     | Non-reactive   |
| Noritrate             | Metronidazole                                     | Non-reactive   |
| Norlutin              | Norethindrone                                     | Non-reactive   |
| Novocain              | Procaine  | Positive for Opiates (OPI, MOP)  |
| Nouvelle              | Norethindrone                                     | Non-reactive   |
| Nycopren              | Naproxen  | Non-reactive   |
| Nystaform             | Chlorhexidine Gluconate                           | Non-reactive   |
| Nystaform             | Hydrocortisone                                    | Non-reactive   |
| Nytol                 | Diphenhydramine                                   | Non-reactive   |
| Opazimes              | Koalin and Morphine Mixture                       | Positive for Opiates (OPI/MOR)   |
| Oramorph              | Morphine  | Positive for Opiates (MOR, OPI)  |
| Orudis                | Ketoprofen  | Non-reactive   |
| Oruvail               | Ketoprofen  | Non-reactive   |
| Otomize               | Dexamethasone                                     | Non-reactive   |
| Otrivine-Anistin      | Antazoline Sulphate                               | Non-reactive   |
| Ovysmen               | Norethindrone                                     | Non-reactive   |
| Ox-pam                | Oxazepam  | Positive for Benzodiazepines (BZD)   |
| Panadol               | Acetaminophen (See also Paracetamol)              | Non-reactive   |
| Panadol Night         | Diphenhydramine                                   | Non-reactive   |
| Paradote              | Paracetamol (Acetaminophen)                       | Non-reactive   |
| Paramol               | Dihydrocodeine                                    | Positive for Opiates (OPI/MOR)   |
| Pavacol-D             | Pholcodine  | Positive for Opiates (MOR, OPI)  |
| Paxil                 | Paroxetine  | Non-reactive   |
| Pediapred             | Prednisolone                                      | Non-reactive   |
| Pediatric BP          | Codeine Phosphate                                 | Positive for Opiates (OPI/MOR)   |
| Penbritin             | Ampicillin  | Non-reactive   |
| Perinal               | Hydrocortisone                                    | Non-reactive   |
| Pethidine             | Meperidine  | Non-reactive   |
| Phenacetin            | Acetophenetidin                                   | Non-reactive   |
| Phenergen             | Promethazine                                      | Non-reactive   |
| Physetone             | Methadone Hydrochloride                           | Positive for Methadone (MTD)   |
| Pipracil              | Piperacillin                                      | Non-reactive   |
| Piriton               | Chlorpheniramine Maleate                          | Non-reactive   |
| Plaquenil             | Hydroxychloroquine                                | Non-reactive   |
| Polycillin            | Ampicillin  | Non-reactive   |
| Ponstan               | Mefenamic Acid                                    | Non-reactive   |
| Ponstel               | Mefenamic Acid                                    | Non-reactive   |
| Powergel              | Ketoprofen  | Non-reactive   |
| Pre lone              | Prednisolone                                      | Non-reactive   |
| Prestim               | Bendrofluazide                                    | Non-reactive   |
| Principen             | Ampicillin  | Non-reactive   |
| Procardia             | Nifedipine  | Non-reactive   |
| Proctofoam            | Hydrocortisone                                    | Non-reactive   |
| Proflex               | Ibuprofen   | Non-reactive   |
| Propetyl              | Pericyazine                                       | Non-reactive   |
| ProSom                | Estazolam   | Positive for Benzodiazepines (BZD)   |
| Prothiaden            | Dothiepin   | Non-reactive   |
| Prozac                | Fluoxetine  | Non-reactive   |
| Psycholept            | Pericyazine                                       | Non-reactive   |
| Pylorid               | Ranitidine  | Positive for Methamphetamine (MET)* *Urinary metabolite(s) only; parent compound is non-reactive |
| Pyrogastrone          | Aluminum Hydroxide                                | Non-reactive   |
| Quinocort             | Hydrocortisone                                    | Non-reactive   |
| Qvar                  | Beclometasone                                     | Non-reactive   |
| Reglan                | Metoclopramide                                    | Non-reactive   |
| Remedeine             | Dihydrocodeine                                    | Positive for Opiates (OPI/MOR)   |
| Remedeine Forte       | Dihydrocodeine                                    | Positive for Opiates (OPI/MOR)   |
| Remedine              | Paracetamol (Acetaminophen) /Codeine Preparations | Positive for Opiates (MOR, OPI)  |
| Restandol             | Testosterone                                      | Non-reactive   |
| Restoril              | Temazepam   | Positive for Benzodiazepines (BZD)   |
| Rivotril              | Clonazepam  | Positive for Benzodiazepines (BZD)   |
| Roferon-A             | Interferon Alfa                                   | Non-reactive   |
| Rohypnol              | Flunitrazepam                                     | Positive for Benzodiazepines (BZD)   |
| Roxanol               | Morphine  | Positive for Opiates (MOR, OPI)  |
| Rozex                 | Metronidazole                                     | Non-reactive   |
| Sarafem               | Fluoxetine  | Non-reactive   |
| Scopoderm             | Hyoscine butylbromide                             | Non-reactive   |
| Seconal               | Secobarbital                                      | Positive for Barbiturates (BAR)  |
| Securopen             | Azlocillin  | Non-reactive   |

| COMPOUND (Trade Name)          | COMPOUND (Generic Name)  | RESULTS  |
|--------------------------------|--|--|
| Senokot                        | Senosides  | Non-reactive   |
| Serax                          | Oxazepam   | Positive for Benzodiazepines (BZD)   |
| Serotulle                      | Chlorhexidine Gluconate  | Non-reactive   |
| Seroxat                        | Paroxetine   | Non-reactive   |
| Severedol                      | Morphine   | Positive for Opiates (MOR, OPI)  |
| Sinequan                       | Doxepin  | Positive for Tricyclic Antidepressants (TCA)   |
| slang name "anabolic steroids" | Testosterone   | Non-reactive   |
| Sofradex                       | Dexamethasone  | Non-reactive   |
| Solpadol                       | Codeine Phosphate  | Positive for Opiates (OPI/MOR)   |
| Solpadol                       | Paracetamol (Acetaminophen) /Codeine Preparations              | Positive for Opiates (MOR, OPI)  |
| Somnite                        | Nitrazepam   | Positive for Benzodiazepines (BZD)   |
| Soneryl                        | Butabarbital   | Positive for Barbiturates (BAR)  |
| Sorbichew                      | Isosorbide Dinitrate   | Non-reactive   |
| Sorbid SA                      | Isosorbide Dinitrate   | Non-reactive   |
| Sorbitrate                     | Isosorbide Dinitrate   | Non-reactive   |
| Spasmonal                      | Alverine Citrate   | Non-reactive   |
| Spasmonal Fibre                | Alverine Citrate   | Non-reactive   |
| Steripod                       | Chlorhexidine Gluconate  | Non-reactive   |
| Stesolid                       | Diazepam   | Positive for Benzodiazepines (BZD)   |
| Stiemycin                      | Erythromycin   | Non-reactive   |
| Strong BP                      | Pholcodine   | Positive for Opiates (MOR, OPI)  |
| Suboxone                       | Naloxone Hydrochlorid/ Buprenorphin Hydrochlorid               | Positive for Buprenorphine (BUP)   |
| Sudafed                        | Pseudoephedrine  | Non-reactive   |
| Sustanon                       | Testosterone   | Non-reactive   |
| Sustiva                        | Efavirenz  | Positive for Cannabinoids (THC)* * Urinary metabolite(s) only; parent compound is non-reactive   |
| Synflex                        | Naproxen   | Non-reactive   |
| Tagamet                        | Cimetidine   | Non-reactive   |
| Tazobactam                     | Piperacillin   | Non-reactive   |
| Tegretol                       | Carbamazepine  | Non-reactive   |
| Temgesic                       | Buprenorphine  | Non-reactive   |
| Tempra                         | Acetaminophen (See also Paracetamol)                           | Non-reactive   |
| Tenben                         | Atenolol   | Non-reactive   |
| Tenben                         | Bendrofluazide   | Non-reactive   |
| Tenif                          | Atenolol   | Non-reactive   |
| Tenorectic                     | Atenolol   | Non-reactive   |
| Tenoret 50                     | Atenolol   | Non-reactive   |
| Tenormin                       | Atenolol   | Non-reactive   |
| Teril                          | Carbamazepine  | Non-reactive   |
| Terra-cortil                   | Hydrocortisone   | Non-reactive   |
| Terra-Cortil                   | Oxytetracycline  | Non-reactive   |
| Terramycin                     | Oxytetracycline  | Non-reactive   |
| Thebacon                       | Pholcodine   | Positive for Opiates (MOR, OPI)  |
| Tiloryth                       | Erythromycin   | Non-reactive   |
| Timodine                       | Hydrocortisone   | Non-reactive   |
| Timonil                        | Carbamazepine  | Non-reactive   |
| Tisept                         | Chlorhexidine Gluconate  | Non-reactive   |
| Topal                          | Aluminum Hydroxide   | Non-reactive   |
| Tramake                        | Tramadol   | Non-reactive   |
| Tranxene                       | Chlorazepate   | Positive for Benzodiazepines (BZD)   |
| Trepergen                      | Co-Phenotrope (atropine/diphenoxylate)                         | Non-reactive   |
| Trexan                         | Naltrexone   | Non-reactive   |
| Trialdone                      | Trazodone  | Non-reactive   |
| Trimovate                      | Oxytetracycline  | Non-reactive   |
| TriNovum                       | Norethindrone  | Non-reactive   |
| Triptafen                      | Amitriptyline  | Positive for Tricyclic Antidepressants (TCA)   |
| Triptafen-M                    | Amitriptyline  | Positive for Tricyclic Antidepressants (TCA)   |
| Trisquens                      | Norethindrone  | Non-reactive   |
| Tryptizol                      | Amitriptyline  | Positive for Tricyclic Antidepressants (TCA)   |
| Tuinal                         | Amobarbital  | Positive for Barbiturates (BAR)  |
| Tylenol                        | Acetaminophen (See also Paracetamol)                           | Non-reactive   |
| Tylenol 3                      | Acetaminophen with Codeine (see also Paracetamol with codeine) | Positive for Opiates (OPI, MOR)  |
| Tylenol 3                      | Paracetamol (Acetaminophen) /Codeine Preparations              | Positive for Opiates (MOR, OPI)  |
| Tylenol Cold (combination)     | Pseudoephedrine  | Non-reactive   |
| Tylenol with codeine           | Acetaminophen with Codeine (see also Paracetamol with codeine) | Positive for Opiates (OPI, MOR)  |
| Tylox                          | Codeine Phosphate  | Positive for Opiates (OPI/MOR)   |
| Tylox                          | Paracetamol (Acetaminophen) /Codeine Preparations              | Positive for Opiates (MOR, OPI)  |
| Ucerax                         | Hydroxyzine Hydrochloride                                      | Non-reactive   |
| Ultram                         | Tramadol   | Non-reactive   |
| Uniroid-HC                     | Hydrocortisone   | Non-reactive   |
| Unisept                        | Chlorhexidine Gluconate  | Non-reactive   |
| Uriflex                        | Chlorhexidine Gluconate  | Non-reactive   |
| Uro-Trainer                    | Chlorhexidine Gluconate  | Non-reactive   |
| Utovian                        | Norethindrone  | Non-reactive   |
| Valclair                       | Diazepam   | Positive for Benzodiazepines (BZD)   |
| Valium                         | Diazepam   | Positive for Benzodiazepines (BZD)   |
| Venotlin                       | Salbutamol   | Non-reactive   |
| Ventide                        | Beclometasone  | Non-reactive   |
| Ventide                        | Salbutamol   | Non-reactive   |
| Ventodisks                     | Salbutamol   | Non-reactive   |
| Vibramycin                     | Doxycycline  | Non-reactive   |
| Vibramycin-D                   | Doxycycline  | Non-reactive   |
| Vicodin                        | Hydrocodone  | Positive for Opiates (OPI/MOR)   |
| Vioform-HC                     | Hydrocortisone   | Non-reactive   |
| Viraferon                      | Interferon Alfa  | Non-reactive   |
| Virormone                      | Testosterone   | Non-reactive   |
| Volmax                         | Salbutamol   | Non-reactive   |
| Volraman                       | Diclofenac Sodium  | Non-reactive   |
| Voltsaid                       | Diclofenac Sodium  | Non-reactive   |
| Voltarol                       | Diclofenac Sodium  | Non-reactive   |
| Wellbutrin                     | Bupropion  | Non-reactive   |
| Welldorm                       | Chloral Hydrate  | Non-reactive   |
| Wellferon                      | Interferon Alfa  | Non-reactive   |
| Xanax                          | Alprazolam   | Positive for Benzodiazepines (BZD)   |
| Xepin                          | Doxepin  | Positive for Tricyclic Antidepressants (TCA)   |
| XTC (slang)                    | Methylenedioxyamphetamine (MDMA)                               | Positive for Methamphetamine (MET) and Ecstasy (MDMA)  |
| Xyloproct                      | Hydrocortisone   | Non-reactive   |
| Zamadol                        | Tramadol   | Non-reactive   |
| Zantac                         | Ranitidine   | Positive for Methamphetamine (MET)* *Urinary metabolite(s) only; parent compound is non-reactive |
| Zestoretic                     | Lisinopril HCl   | Non-reactive   |
| Zestril                        | Lisinopril HCl   | Non-reactive   |
| Zidoval                        | Metronidazole  | Non-reactive   |

| COMPOUND (Trade Name) | COMPOUND (Generic Name) | RESULTS                            |
|-----------------------|-------------------------|------------------------------------|
| Zineryl               | Erythromycin            | Non-reactive                       |
| Zita                  | Cimetidine              | Non-reactive                       |
| Zoloft                | Sertraline              | Non-reactive                       |
| Zonivent              | Beclometasone           | Non-reactive                       |
| Zoton                 | Lansoprazole            | Non-reactive                       |
| Zyban                 | Bupropion               | Non-reactive                       |
| Zydol                 | Tramadol                | Non-reactive                       |
| Zymoet                | Metronidazole           | Non-reactive                       |
| Zyprexa               | Olanzapine              | Non-reactive                       |
|                       | Aminopyrine             | Non-reactive                       |
|                       | Aprobarbital            | Positive for Barbiturates (BAR)    |
|                       | Barbital                | Positive for Barbiturates (BAR)    |
|                       | Butethal                | Positive for Barbiturates (BAR)    |
|                       | Ethylmorphine           | Positive for Opiates (OPI/MOR)     |
| Vick's Inhaler        | l-Methamphetamine HCL   | Positive for Methamphetamine (MET) |
|                       | Progesterol             | Non-reactive                       |